

Product information: 20504

fast-acting disinfection

- ready-to-use alcoholic fast-acting disinfectant
- extremely fast effectiveness, also against tuberculosis
- aldehyde-free
- skin-friendly



## RHEOSEPT-SD universal

Fast-acting alcoholic disinfectant

### Product description

Aldehyde and phenol-free disinfectant for professional use in practices, clinics and care facilities with a large range of action and short reaction times. High material compatibility through reduced alcohol content, very skin-friendly, dermatologically tested.

### Fields of application

For prevention and control of infectious diseases. Cleaning and disinfection of all water and alcohol-resistant surfaces of non-invasive medical devices, small appliances, medical/dental equipment.

### Dosing instructions

Pour the solution on a dry cloth and wet the surface to be disinfected by evenly wiping over it. Direct spraying, where the solution is sprayed directly on the surface, should only be used if absolutely necessary. With both methods, make sure that the surface to be disinfected is fully and sufficiently wetted. The treated surface dries without

residues. Please see the product information for reaction times. Remove any dirt and moisture from surfaces before treatment.

### Spectrum of activity

Bactericidal (incl. TB), fungicidal, virucidal (enveloped viruses such as HIV, HBV, HCV, influenza, and also NORO).

### Composition

100 g of solution contain: 25.9g ethanol, 11.8g isopropyl alcohol, 0.2g polyaminopropyl biguanide.

### Physico-chemical data

- Flash point >23°C

## Labelling in accordance with the EC-directives

- H319 - Causes serious eye irritation.
- H226 - Flammable liquid and vapour.
- H412 - Harmful to aquatic life with long lasting effects.
- P210 - Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- P280 - Wear protective gloves/protective clothing/eye protection/face protection.
- P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- P337 + P313 - If eye irritation persists: Get medical advice/attention.
- P370 + P378 - In case of fire: Use foam, powder or carbon dioxide to extinguish.
- P403 + P235 - Store in a well-ventilated place. Keep cool.
- EUH208 - Contains polyhexamethylene biguanide hydrochloride. May produce an allergic reaction.

## Forms of delivery

REF	Package size	Packaging unit
20504352	RHEOSEPT-SD universal bottle 1000 ml	carton
20504100	RHEOSOL-SD universal canister 10 Litre	canister

## Accessories

REF	Package size	Packaging unit
48006003	Sprayer Maxi bulk	piece
48020002	Canister pump plug 51 mm, 30 ml pump stroke canister pump for 5 and 10-litres-canister blue or white with UN-code Riser pipe length 300 mm	piece
48038005	Canister key plug 51	piece
48017002	Canister tap for 5 and 10-litres-canister with plug 51	piece
48029002	Bottle holder for 4 bottles à 1000 ml	piece
48029003	Canister hook for 10 litres canister for wall fastening	piece
48029001	Canister holder for 10 litres canister	piece
48036001	Protective gloves yellow, size S, package with 12 pairs 100 % natural latex, lined with cotton, rolled edging, suitable for the food processing industry	package
48036006	Protective gloves yellow, size M, package with 12 pairs 100 % natural latex, lined cotton, rolled edging, suitable for the food processing industry	package
48036005	Protective gloves yellow, size L, package with 12 pairs 100 % natural latex, lined cotton, rolled edging, suitable for the food processing industry	package
48036003	Protective gloves yellow, size XL, package with 12 pairs 100 % natural latex, lined cotton, rolled edging, suitable for the food processing industry	package
48036002	Protective gloves blue, 1 pair Latex, size 10, acid and alkaline-resistant (30 cm long) package with 12 pairs	package

## Certificate and information

For additional information on this product, please see → [www.rheosol.com](http://www.rheosol.com) – For specific questions or information about expert reports, please contact our office staff: Telephone: +49 2241 - 3923-70 e-mail: [info@rheosol.de](mailto:info@rheosol.de).

## Environmental information

Wachendorff-Chemie GmbH produce under application of advanced, safe and environmentally friendly procedures and in compliance with high quality standards.