


Product information: 20512

wiping disinfection

- ready to use and fast acting disinfection, alcohol-free
- with good cleaning capacities and pleasant scents
- also applicable as foam
- aldehyde-free
- extremely fast bactericidal, fungicidal and viricidal activity
- effective against influenza, noro and MRSA
- VAH listed
- skin-friendly - was awarded highest marks when dermatologically tested

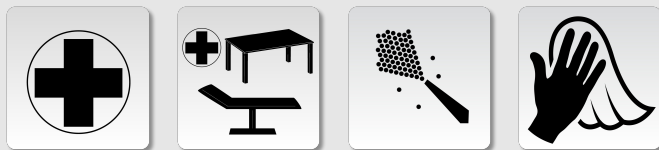

CE
0481

RHEOSEPT-WD plus

Fast-acting, alcohol-free and ready to use wiping disinfectant

Product description

Ready to use and alcohol-free fast-acting disinfectant with good cleaning capabilities and pleasant scents for wiping disinfection and surface cleaning of water-resistant surfaces (as well acrylic surfaces) in the medical area. In ready-to-use concentration. Fully effective already after just 1 minute. Listed with the VAH (Verband für Angewandte Hygiene e.V.) acc. to the guidelines of the DGHM (Deutsche Gesellschaft für Hygiene und Mikrobiologie e.V.). Spectrum of activity: bactericidal, fungicidal, viricidal (HIV, HBC, HCV, noro, rota, vaccinia, BVDV). This medical product is CE certified (CE 0481).

Fields of application

For prevention and control of infectious diseases. Fast-

acting disinfection by means of a wipe process for all water-resistant surfaces of medical products such as device surfaces, seats, examination tables, pans as well as sunbeds. Suitable for materials such as plastics, lacquers, ceramics, acrylic glass etc. Special suitability assessments for plexi glass and interior fitting of ambulance vehicles are available.

Dosing instructions

For cleaning and wiping disinfection apply product undiluted on the surface to be disinfected and leave it on for minimum one minute. Please pay careful attention to the residence time according to the application area. When using the RHEOFIX-wischBox WD-yellow, the maximum lifetime of 4 weeks must be considered. If visibly pre-contaminated pre-clean the surface firstly with RHEOSEPT-WD plus. According to the application area of the medical

product the surface is to rinse or wipe carefully after disinfection in order to remove product residues. The product can also be used with RHEOSOL-Foamer Maxi or a pump foam bottle. Do not combine with other cleaning agents or disinfectants.

Spectrum of activity

Spectrum of activity: bactericidal, fungicidal, viricidal (HIV, HBC, HCV, noro, rota, vaccinia, BVDV).

Contact times

Dosing specifications	Dosing details	Remark
contact time 60 seconds	undiluted	bactericide/fungicide EN 1276, EN 13697, EN 1650
contact time 5 minutes	undiluted	bactericide/fungicide (VAH) low load/mechanics
contact time 60 seconds	undiluted	enveloped viruses (HIV, HBV, HCV, BVDV, vaccinia, influenza) EN 14476
contact time 30 minutes	undiluted	norovirus (MNV) EN 14476
contact time 60 seconds	undiluted	rotavirus
contact time 5 minutes	undiluted	polyomavirus SV40

Composition

100g contain: 0.25g dialkyldimethylammoniumchloride, 0.25 g benzalkoniumchloride, 0.25 g alkyldimethyl(ethylbenzyl)ammoniumchloride. Further ingredients: scents, limonenes.

Physico-chemical data

- pH-value (concentrate) 7
- pH-value (application) approx. 7

Special notes

This product is no hazardous material according to the Ordinance on Hazardous Substances. The valid preventive measures for the handling with liquid chemicals should be attended anyhow. Keep container tightly closed and protect from sunlight. Attend charge number and expiry date as separate imprint on package. Do not spray product. Use foam valve if applicable.

Forms of delivery

Product code	Package size	Packaging unit
20512352	RHEOSEPT-WD plus bottle 1000 ml	carton
20512001	RHEOSEPT-WD plus refill set carton	carton
20512050	RHEOSEPT-WD plus canister 5 litres	canister
20512100	RHEOSEPT-WD plus canister 10 litres	canister
20512002	RHEOFIX-Wiping box WD-yellow 2 buckets with each 100 dry wipes and 2 bottles RHEOSEPT-WD plus bottle 1000 ml	carton
71004002	RHEOFIX-wipeBox refill cloth dry 100 dry cloths	carton

Accessories

product code	package size	packaging unit
48038005	Canister key plug 51	piece
48017002	Canister tap for 5 and 10-litres-canister with plug 51	piece
48020002	Canister pump plug 51mm, 30 ml pump stroke canister pump for 5 and 10-litres-canister blue or white with UN-code Riser pipe length 300 mm	piece
48029002	Bottle holder for 4 bottles à 1000 ml	piece
48029006	Bottle holder for 6 bottles a 1000 ml stainless steel	piece
48029003	Canister hook for 10 litres canister for wall fastening	piece
48006009	Foamer Maxi bulk piece	piece

Certificate and information

For additional information on this product, please see → www.rheosol.com – For specific questions or information about expert reports, please contact our office staff: Telephone: +49 2241 - 3923-70 e-mail: info@rheosol.de.

Admission

Listed with the VAH (Verbund für Angewandte Hygiene e.V.) acc. to the guidelines of the DGHM (Deutsche Gesellschaft für Hygiene und Mikrobiologie e.V.). This medical product is CE certified (CE 0481).

RHEOSEPT®

Environmental information

Wachendorff-Chemie GmbH produce under application of advanced, safe and environmentally friendly procedures and in compliance with high quality standards.

RHEOSOL®

Wachendorff-Chemie GmbH • Langbaughstr. 15 • DE 53842 Troisdorf
☎ +49 2241 3923-0 📠 +49 2241 3923-90 ✉ info@rheosol.de 🌐 <http://www.rheosol.com>