RHEOSEPT

Product information: 20524

fast-acting disinfection

- ready to use and alcoholic fast-acting disinfection
- extremely fast bactericidal and viricidal activity in case of TBC
- aldehyde-free, without covering perfume
- fast residue-free drying
- skin-friendly was awarded highest marks when dermatologically tested
- VAH listed













RHEOSEPT-SD plus

Alcohol-based fast-acting disinfectant

Product description

Aldehyde-free wiping disinfection for fast-acting and extensive disinfection measures with pleasant scents. The product dries with almost no residue. Spectrum of activity: bactericide incl. TBC, fungicide, viricide (HIV, HBV, HCV, noro, vaccinia, BVDV). This medical product is CE certified (CE 0481).

Fields of application

For fast-acting disinfection of all water and alcohol resistant surfaces of medical products (not acryl or plexiglass). For the institutional use in surgeries, hospitals and maintenance institutions.

Dosing instructions

Detach surface from contamination and wetness before disinfection. Apply the liquid and ready to use solution on a cloth and wipe the dry surface evenly. The surface is to wet

sufficiently. Use a direct spray procedure only when not inevitable. The residence time is to last minimum 1 minute. Do not use on acrylic (plexiglass). Proteins decrease the disinfection effect considerably.

Spectrum of activity

Bactericide incl. TBC, fungicide, viricide (HIV/ HBV/ HCV/ Noro/ Vaccinia/ BVDV).

RHEOSEPT®

Contact times

Dosing specifications	Dosing details	Remark
contact time 60 seconds	undiluted	bactericide/fungicide EN 1276, EN 13697, EN 1650
contact time 5 minutes	undiluted	bactericide/fungicide (VAH) low load/mechanics
contact time 60 seconds	undiluted	enveloped viruses (HIV, HBV, HCV, BVDV, vaccinia, influenza) EN 14476
contact time 60 seconds	undiluted	norovirus (MNV) EN 14476
contact time 60 seconds	undiluted	mycobacteria (TBC)

Composition

100g contain: 37g ethanol, 13g propan-2-ol, 0.08g alkylbiguanide.

Physico-chemical data

> Flash point >21°C

Labelling in accordance with the EC-directives

- > H319 Causes serious eye irritation.
- > H226 Flammable liquid and vapour.
- > H412 Harmful to aquatic life with long lasting effects.
- P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- P280 Wear protective gloves/protective clothing/eye protection/face protection.
- P303 + P361 + P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water [or shower].
- P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- > P337 + P313 If eye irritation persists: Get medical advice/attention.
- > P403 + P235 Store in a well-ventilated place. Keep cool.
- > EUH208 Contains polyhexamethylene biguanide hydrochloride. May produce an allergic reaction.

Specific advices

Note batch number and expiry date separately printed on the packaging. Only for professional users. A rinsing with water afterwards is generally not necessary. The product dries with almost no residue. Depending on the application area of the medical product the surface is to rinse carefully with water after the disinfection in order to remove possible product residues. Always store closed and under dry, cool conditions protected from direct sunlight.

Forms of delivery

Product code	Package size	Packaging unit
20524310	RHEOSEPT-SD plus spray bottle 200 ml	carton
20524340	RHEOSEPT-SD plus bottle 500 ml	carton
20524352	RHEOSEPT-SD plus bottle 1000 ml	carton
20524050	RHEOSEPT-SD plus canister 5 litres	canister
20524100	RHEOSEPT-SD plus canister 10 litres	canister

Accessories

48006001	Spray bottle Maxi 1000 ml with sprayer empty	piece
48006003	Sprayer Maxi bulk	piece
48038005	Canister key plug 51	piece
48017002	Canister tap for 5 and 10- litres-canister with plug 51	piece
48020002	Canister pump plug 51mm, 30 ml pump stroke canister pump for 5 and 10-litres- canister blue or white with UN-code Riser pipe length 300 mm	piece
48029002	Bottle holder for 4 bottles à 1000 ml	piece
48029006	Bottle holder for 6 bottles a 1000 ml stainless steel	piece
48029003	Canister hook for 10 litres canister for wall fastening	piece

Certificate and information

For additional information on this product, please see → www.rheosol.com – For specific questions or information about expert reports, please contact our office staff: Telephone: +49 2241 - 3923-70 e-mail: info@rheosol.de.

Admission

This medical product is CE certified (CE 0481).





Environmental information

Wachendorff-Chemie GmbH produce under application of advanced, safe and environmentally friendly procedures and in compliance with high quality standards.

